

Introducing: DynaVox T-10 and Compass

Nuala South, MA, CCC-SLP
AAC Consultant, DynaVox Mayer Johnson
405-822-5799

Nuala.south@dynavoxtech.com

1



Learner Objectives:

- Participants will learn how to select an appropriate page set based on the users communication ability level.
- Participants will learn how to program pagesets within the Compass software.
- Participants will become familiar with available resources at mydynavox.com for implementation and programming of the DynaVox T10 and Compass software.

2



Hardware overview



3



Hardware Overview

Front:	1 Charging Indicator: Lit amber when the battery is charging. Turns blue when the battery is fully charged.
	2 Power Button: To turn the T10 on, press and hold until the splash screen appears. Press and release to put to sleep. To shut down the T10, press and hold until the shutdown menu appears.
	3 Front-facing Camera: Takes photographs of people and objects in view of the T10 screen - for example, self portraits.
	4 Ambient Light Sensor: Adjusts the screen brightness based on the surrounding light.
Top	5 Speakers: Dual speakers provide an extensive range of volume.
	6 Volume Rocker Switch: Press on the + side of the switch to increase volume. Press on the - side of the switch to decrease volume.
Right Side	7 Speaker/Headphone/External Microphone Jack: Connect an external speaker, headphones for private audio, or an external microphone.
Back	8 Rear-facing Camera: Use for taking custom photos to share with others or to put on your communication pages.
	9 Lanyard connections: Provide attachment points for the T10 strap.
Bottom	10 Charger Port: Plug in the adapter/charger to charge the battery.
	11 Micro USB Port: Connect to a micro-USB to full-size USB cable adaptor. (DynaVox does NOT recommend the use of a USB hub.)
	12 30-Pin Dock Connector: (for future use)
	13 Micro HDMI Port: Connect an external monitor to your T10.
	14 Built-in Microphone: Record speech and custom sounds.

4



Software

- DynaVox Compass
 - New software/app
 - T10 is Android OS
 - Ipad 2 or newer, or Ipad mini
 - Windows computer or tablet
 - Editing software for Windows
 - MyDynaVox.com
 - Cloud based- **For Funded Devices Only**
 - Contains all page sets, software, [videos](#), [tutorials](#), [resources](#), [curriculum support](#), etc.
 - Let's Take a Look!

5



Software

- InterAACt Philosophy
 - All DynaVox page sets reflect the commitment to InterAACt Philosophy
 - Balance of Efficiency and Precision
 - ToolBar-Quickfires, Quick Phrases, Core, Keyboard, Topic Messages
 - New tools: Whiteboard, Rating Scale, Behavior Supports
 - Three Core vocabulary options, 'Core', 'Word Power', and 'Gateway'.
 - Decreased need for 'customizing' due to significantly increased vocabulary availability/content

6



Ability Levels

Ability Levels within the NavBar & Master Page Pagerset

The NavBar and Master Page Pagerset in the Dynavox Communicator™ software have been developed to ensure the right communication tools at the optimal arrangement for you. Each Ability Level or communication page layout will support communication needs while taking advantage of the skills that you already have to encourage use and progress.

5 Topics Topic Messages V5-4, Grid-8	10 Topics Topic Messages V5-8, Grid-12	20 Topics Topic Messages V5-12, Grid-16	Unlimited Topics Topic Messages V5-12, Grid-20	Unlimited Topics Topic Messages V5-12, Grid-24
12 QuickFires	20 QuickFires	30 QuickFires	48 QuickFires	48 QuickFires
QuickFires Picture, Greeting, Feedback	QuickFires Picture, Greeting, Feedback	QuickFires Picture, Greeting, Feedback, Question	QuickFires Picture, Greeting, Feedback, Question, Answer	QuickFires Picture, Greeting, Feedback, Question, Answer, Near Choice
Word Lists 5 words per category	Word Lists 10 words per category	Word Lists 20 words per category	Word Lists 30 words per category	Word Lists 30 words per category
Core Step 1	Core Step 2	Core Step 3	Core Step 4	Core Step 5
Keyboard: Alphabet	Keyboard: Alphabet	Keyboard: QWERTY	Keyboard: QWERTY	Keyboard: QWERTY
Positive Behavior Supports & Scripts	Positive Behavior Supports & Scripts	Positive Behavior Supports & Scripts	Positive Behavior Supports & Scripts	Positive Behavior Supports & Scripts
EMERGENT LEVEL 1	EMERGENT TRANSITIONAL LEVEL 2	CONTEXT DEPENDENT LEVEL 3	INDEPENDENT TRANSITIONAL LEVEL 4	INDEPENDENT LEVEL 5

Active on an optional: Networks, Whiteboard, Collages, Word Prediction, Core Steps 1-8

7



PowerHouse

- Find in your Power House:
 - Don't!
 - Mom!
 - I need to use the bathroom.
 - I like that
 - Art class
 - Zebra

8



Page Sets

- Nav Bar:
 - Unique design; allows you to access any Topic without switching between pages by swiping
 - "Flat" navigation
 - Choose topics at beginning; add and delete as needed
 - "Powerhouse" remains to the side at all times, stable positions
 - Visual scenes or grid (choose at beginning!)
 - Each tool can be customized to the client's needs
 - Data Linked Vocabulary



9



Page Sets

- Master Page:
 - Consistent home (i.e., the Master Page) with links to all of your Topics and communication tools.
 - Visual scenes or a more traditional grid of messages for each topic.
 - Stable grid for Keyboard users.
 - Most Similar to Series 5 pages
 - Power house consistently on right side



10



Settings

- Nav Bar
- Master Page




- Open the Settings Menu if the Toolbar is Visible:
Simply select the Settings button on the Toolbar. The Settings menu will open.
- Open the Settings Menu if the Toolbar is Not Visible:
1. Select the Left button in the top right corner of your screen.
Bar: Bottom
- If the Toolbar is Hidden, a dialog will open presenting you with two options.
- Select the Settings button. The Settings menu will open.



13

Settings Menu

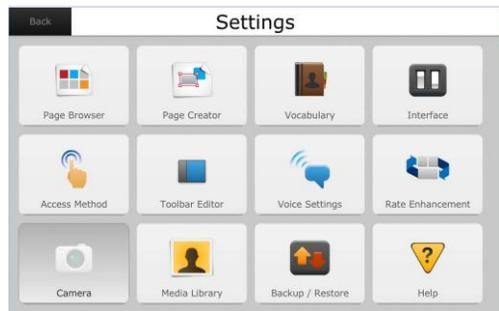

14



Voice Settings

- Change voice
 - Touch down arrow- choose voice
- Volume & Rate
 - Slider button
- Manage Pronunciation
 - New Phoneme Exception
 - Use the example sounds to write out the way the word should sound



15

Camera and Media Library




- Use this camera to take pictures;
 - **it will store them in your media library
- After taking picture, click "New Picture" to save it
- Can use them across the page set- can explore them in the media library
- Pictures can be uploaded via the USB drive.



18

Customizing

- Quick Phrases
 - Edit button (Hamburger)
 - Edit Page

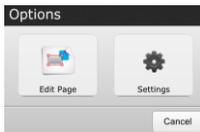

20

Customizing

- Touch button you want to customize
- Edit



21

Customizing


- Label
- Symbol
 - Select
 - "Search" for symbol
 - "Image" for pictures you have taken with the camera
- Do not edit action



22

Customizing Topics

- If you are using Visual Scenes / Hotspots
 - Each hotspot opens to a new popup
- To Customize Popups
 - Navigate to the topic you want to customize
 - Touch "Edit"
 - File- open page (choose whichever you want to program)
 - Customize buttons
- Save!!!

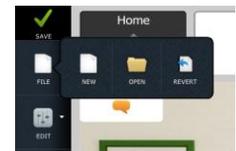

23

Customizing YOUR Master Page


- Go to Classroom, topic words, classmates, add 3 classmates names
- Add 3 age appropriate phrases to "Quick Phrases" (Modify "My name is...")

24



Customizing NavBar

- Base User
 - Topics already chosen
 - Add topics:
 - Settings
 - Toolbar Editor
 - Touch "Topics"
 - "Edit Items"
 - Manage
 - Scroll to find topics
 - Touch desired topics
 - When done, touch "Show"
 - Done

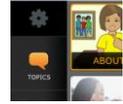

25



NavBar




26



NavBar

- Customizing Topic Buttons
 - Edit/Hamburger
 - Highlight the Tab you want to change
 - THEN touch below to grab the box of buttons (not the single button)
 - Touch edit

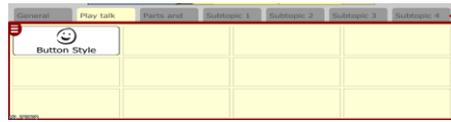

27



NavBar

- Click "edit vocabulary"



28



NavBar

- Choose button
- Customize label
- Browse to modify symbol (can use media browser)
- Message
- Save



29



NavBar

- Reorder Buttons
 - Manage
 - Select button
 - "order" (essentially swap)
 - Move first, last or right/left
 - Done



30



Pronunciation Exception

- Manage Pronunciation
 - New Phoneme Exception
 - Use the example sounds to write out the way the word should sound




32



Backup Page Set

- Insert MicroUSB converter into port on bottom of device
- Insert USB flashdrive into converter
- Settings-Backup/Restore
- Backup Locally

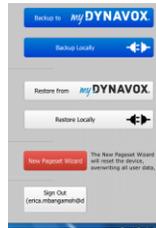

33



Backup Page Set

- Hit the "Up" arrow until you see drives
- Select your flashdrive (USB)
- Touch "Select"
- Type in the name on the prompt
- Touch "OK"



34



Support



Local Consultants
Your local consultant provides device demonstrations and product training.



Funding Support
A team of specialists who work with our clients to secure funding from Medicare, Medicaid and other insurance sources.



Technical Support
Available toll-free to assist professionals and clients. With the DynaVox T10, Vmax, Xpress, and Maestro, our technicians can even remotely access the device to help you begin communicating again quickly.

Connect with us:

CONNECT WITH US



- **Want to know what all the buzz is about?** Join DynaVox Mayer-Johnson on Twitter and Facebook to learn about our newest products, promotions and giveaways.
- **Interested in tutorials, home videos and news updates?** Visit us on YouTube and Vimeo.

www.mydynavox.com

