

## EZ Carry Handle

---

The EZ Carry Handle will make it much easier for people with arthritis or weakened wrist joints to carry any type of bag.

### Materials:

- 1 – Wooden Dowel or PVC pipe (6 in. length, ½ inch diameter)
- 1 – Screw (½ inch)
- 1 – Over the Door Hook
- Screwdriver

### Directions:

1. Pre-drill a small hole in the middle of the dowel
2. Line up the hook with the pre-drilled hole
3. Use screwdriver to tighten screw into dowel



**Purchase Price: \$5.75**



**Make-N-Take Price: \$2.50**

### Helpful Hints:

No drill? Ask at a home improvement store to help pre-drill the holes or use a hammer and nail to start a small hole for the screws.

When purchasing the Over the Door Hook look for one that has a pre-drilled hole at the top to make it much easier to assemble.