

Make 'n Take Sheet

Discover How Easy It Is To Make Your Own Assistive Technology!

ABLE
Tech

Ring on a Mouse

By adding the loop of the ring to the mouse, it helps individuals that have a hard time keeping their finger on the left trigger.

Warning: May be a choking hazzard for kids 5 and under

Materials:

Plastic Ring
Scissors
Super Glue/Hot Glue Gun/
Glue
Sand paper (optional)



Make-N-Take Price: \$1.00

Directions:

1. Cut the decorative piece of the ring off, leaving just the loop.
2. (OPTIONAL) Use the sandpaper and smooth out the surface being attached to the mouse.
3. Adhere the part of the loop that was previously attached to the decorative piece to the mouse.
4. Let the adhesive dry before using.

Helpful Hints: You can usually find the little fancy decorative rings at the dollar store or at a party store.

Oklahoma ABLE Tech, the statewide Assistive Technology Act Program, is funded through the Administration for Community Living of the U.S. Department of Health and Human Services, and maintains coordination and collaboration efforts with partners throughout the State of Oklahoma.

To contact Oklahoma ABLE Tech call 1-800-257-1705 or INFO-line 1-888-885-5588