

Make 'n Take Sheet

Discover How Easy It Is To Make Your Own Assistive Technology!

iPad Stand

An iPad Holder provides a way for the iPad to stand up on its own.




Materials:

- 2 pieces of Stratacore, 12X12
- 1 roll Double sided tape
- 1 Utility knife
- 3 pc Velcro
- 1 Clear Corner Guard
- 1 Package of "Hold-it"
- Scissors

Directions:

1. Score 1 inch from the end—against the grain using a utility knife
2. Flip piece of stratacore over and score in the middle
3. Score the other end, 2 inches from the end
4. Bend stratacore along the score lines
5. Apply double sided tape to the 1 inch score end
6. Apply loop Velcro to the 2 inch score end
7. Place two strips of Velcro in the middle of mounting platform
8. Attach the folding piece to the mounting base
9. Fold over and attach to the mounting base
10. Attach plastic lip (corner guard) to end

For more a step-by-step instructional video, please visit <https://www.youtube.com/watch?v=j1pTvtAQjUU&feature=youtu.be>

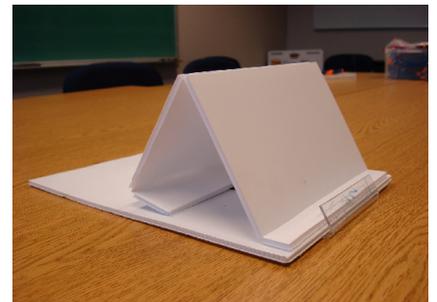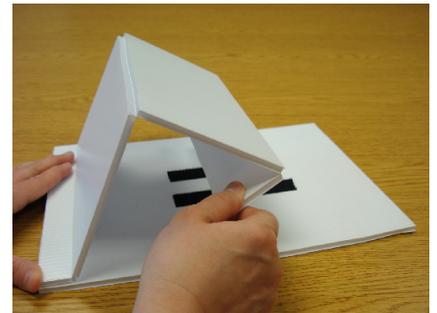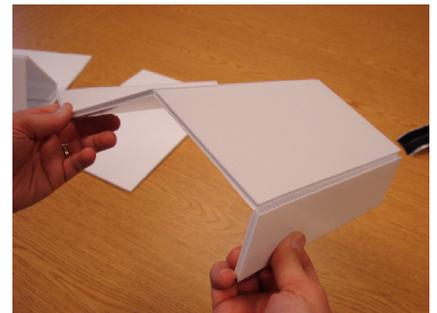

Oklahoma ABLE Tech, the statewide Assistive Technology Act Program, is funded through the Administration for Community Living of the U.S. Department of Health and Human Services, and maintains coordination and collaboration efforts with partners throughout the State of Oklahoma.

To contact Oklahoma ABLE Tech call 1-800-257-1705 or INFO-line 1-888-885-5588