



**STATE OF OKLAHOMA  
BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE**

Dr. Kenneth E. Blick, Ph.D.  
Chairman

Administrative Office at:  
Board of Tests for Alcohol and Drug Influence  
Post Office Box 36307  
Oklahoma City, Oklahoma 73136-2307

J. Robert Blakeburn  
State Director of Tests  
for Alcohol and Drug Influence

April 10, 2008

**ACTION NO. 08-02 BY THE STATE DIRECTOR OF TESTS FOR ALCOHOL AND DRUG  
INFLUENCE**

Under authority of 47 O.S. Supp.2000, Section 751-761 and OAC 40:1-1-3. and 40:30-1-3. Breath-alcohol analysis with the Intoxilyzer Model 5000-D; July 1, 1997, promulgated by the Board of Tests for Alcohol and Drug Influence, the following official **ACTION** is hereby taken by the State Director of Tests for Alcohol and Drug Influence.

**OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS  
WITH THE MODEL 5000-D INTOXILYZER WITH KEYBOARD AND EXTERNAL PRINTER**

- (1) The State Director of Tests for Alcohol and Drug Influence hereby prescribes and approves the attached "**OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS WITH THE MODEL 5000-D INTOXILYZER WITH KEYBOARD and EXTERNAL PRINTER**" for the analysis of alcohol in specimens of breath by means of the Model 5000-D Intoxilyzer with keyboard and external printer, in keeping with provisions of OAC 40:30-1-3 promulgated by the Board of Tests for Alcohol and Drug Influence. The **OPERATING PROCEDURE** is attached hereto and is hereby incorporated in this record of official **ACTION**.
- (2) All testing completed on or after April 10, 2008, that has followed the procedures required by Action 08-02, shall be deemed to have been performed in accordance with the Board of Test rules.
- (3) This **ACTION** by the State Director of Tests for Alcohol and Drug Influence shall take effect on and after April 10, 2008.

A handwritten signature in cursive script, reading "J. Robert Blakeburn".

J. Robert Blakeburn  
State Director of Tests  
for Alcohol and Drug Influence



State of Oklahoma  
Board of Tests for Alcohol and Drug Influence  
Office of the State Director of Tests for Alcohol and Drug Influence  
Post Office Box 11415  
Oklahoma City, Oklahoma 73136-0415

Breath-Alcohol tests administered by means of the Model 5000-D Intoxilyzer under the provisions of 47 O.S. Supp.2000, Section 751-761 and OAC 40:30-1-3., July 1, 1997, promulgated by the Board of Tests for Alcohol and Drug Influence, shall be performed in accordance with the following **OPERATING PROCEDURE** established by the State Director of Tests for Alcohol and Drug Influence.

**OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS  
WITH THE MODEL 5000-D INTOXYLIZER WITH KEYBOARD AND EXTERNAL PRINTER**

**A. SUBJECT PREPARATION**

1. **OAC 40:30-1-3. (c) (1)** The subject's mouth shall be examined for presence of any loose material(s), or foreign substance(s), or any such substance(s) and this material or substances shall be removed. Metal, porcelain or hard plastic items need not be removed.
2. Ascertain that, **at least fifteen (15) minutes prior to collection of the first breath specimen**, the subject has not ingested any substance at all, vomited, eaten, or smoked. Such observation may be carried out by the breath-alcohol analysis Operator or Specialist or by any other qualified person.

**B. TESTING SEQUENCE**

2. Press the Green Start button on the Intoxilyzer and follow the instructions displayed. **Note:** Remove a new sealed mouthpiece from its clear bag and insert it into the end of the Breath Tube before the first test is administered.
3. Upon completion of a test, the printer will print the testing data on the Officer's Affidavit.

**C. ROUTING OF DOCUMENTS**

1. The printer will print two copies. Remove the Affidavit from the external printer and sign the copy to be forwarded to the Department of Public Safety. Make two copies of the FRONT:
2. Forward the signed printout to the Department.
3. Deliver the other printout to the person tested.
4. Deliver one of the copies to the prosecutor and retain one for your records.  
**KEEP YOUR COPY.**

**D. COMPLETION OF TEST**

5. Leave the Intoxilyzer, Simulator, and Printer on and ready for the next test.

**E. REPETITION OF BLANK, SUBJECT, AND CONTROL TESTS**

6. If any other programmed message interrupts the test or the difference between the breath-alcohol concentrations of the subject's BREATH TEST 1 and BREATH TEST 2 exceeds 0.03 g/210 L, repeat the entire procedure in Section B. Attach the program message to the affidavit. **DO NOT THROW ANY PRINTOUT AWAY.**